


PACKAGE CONTENTS

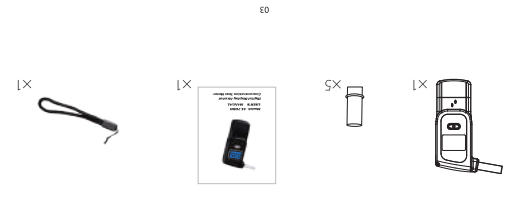

1. One AT7100 digital display alcohol concentration detector.
 2. One User Manual.
 3. Three AAA alkaline batteries. (not included).
 4. 5 air-blowing pipes.
 5. One sling.

●After opening the package, please check the parts and components below. If any one is absent, please contact its dealer.
 ●Accessories

Preparation before using

BAC	Dose-Specific Effects
0.020-0.039%	Generally no loss of coordination. Slight joy or loss of shyness. Depressant effects are not clear.
0.040-0.069%	General feelings of well-being, relaxation, shyness is reduced and sensation of warmth, joy. Some minor impairment of reasoning and memory. Lowering of caution. Driving skills may be impaired.
0.070-0.099%	Slight impairment of balance, speech, vision, reaction time, and hearing. Joy, judgment and self-control are reduced. Caution, reason and memory are impaired. Driving skills are always impaired.
0.100-0.129%	Major impairment of motor coordination and loss of good judgment. Speech may be slurred. Balance, vision, reaction time and hearing will be impaired. Joy.
0.130-0.159%	Gross motor impairment and lack of physical control. Blurred vision and major loss of balance. Joy reduced and state of feeling unwell begins to appear.
0.160-0.190%	Anxiety, restlessness predominates. Nausea may appear. Appearance of a 'stopy drunk'.

Declaration


1.Regardless of whether the product is used,manufacturers and vendors will not bear any responsibility for the violations,for example:drunk driving and high-altitude operations driving.
 2.Each person reacts differently to alcohol,so the test results are for reference only, which cannot serve as the criteria in determining the outcome.

Common Faults and Solutions

Faults	Possible Faults Reason	Solution
Power on failing	Put batteries incorrectly	Installing batteries in correct polarity
	Very low power alkaline batteries	Replacing the new batteries
Blowing no response	Circuit Fault	Contacting after-sale service
	Warm up for more times	Warm up for more times
Blowing no result	Residual Gas in tester	Warm up again
	Not blowing in scheduled time	Blowing during "BLOW" countdown time
No blowing with test result	Residual gas in tester	"Warm up" for more times
Different result after several testing	Testing as soon as minutes of drinking	Testing after 20 minutes of drinking
	Blowing time different	Blowing time different 3-5 seconds

Test

●The "START" symbols displayed when work-up is succeed, and the countdown is started that indicate the tester is in testing mode.

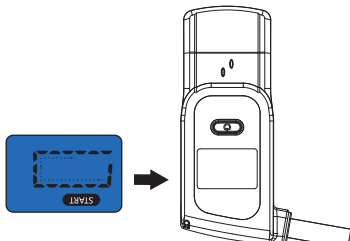

●Near and blow into the breath inhaler for 3-5 seconds.
 ●The analysis of...


Auto power off

1.After the result be display for about 10 seconds, the "off" symbol will appear 2 seconds and the alcohol tester be turned off.
 2.If work up is defeated, the tester will automatically turned off.
 3.Long press the POWER button to turn on the product and long press it again to turn off in any mode.
 After displaying the testing result, click POWER button to re-activate the TESTING mode directly, no need to wait till it shut off to click the POWER button.

Restart mode

●Read the tester result from the alcohol tester lcd.


Unit conversion

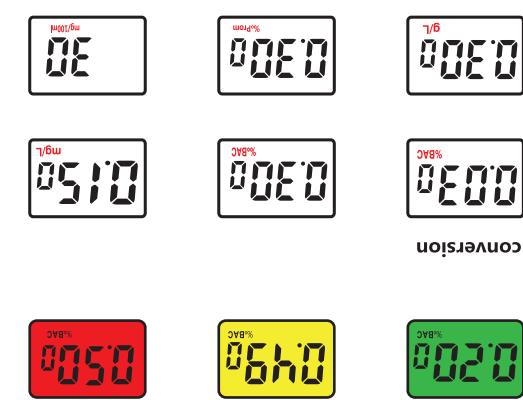

Precautions

- Thank you for selecting and purchasing AT7100 digital display alcohol concentration test meter. Please read this manual carefully to ensure that you can use the product correctly.
- This device is a healthcare product; it is intended to control overdrink of alcohol so as to keep you fit, through testing the alcohol in your breath and calculating the content of alcohol in the blood via microcomputer. The product is applicable to family healthcare as well as for reference in determining whether the driver is driving after drinking.
- The test result may be affected by a number of aspects such as environmental temperature, various gases in the air and smoking, etc. the result shall, therefore, be only for reference and shall not be regarded as the justification of driving after drinking.
- No matter whether you have consumed this product, its manufacturer and appropriate agents shall be not responsible for any traffic accident arisen due to ethanol or other accidents happened under the related regulations and rules.
- As people's breath contains some fermentable gases (from stomach and intestines), so some gases may even be tested out under no drink circumstance.
- When after drinking, it is recommended to last after 20 minutes, it is because 20 minutes is approximately for alcohol to be absorbed into blood from the digestive organs, and residual alcohol remaining in the mouth takes this long to dissipate.
- For accurate result, do not repeat neat testing with in 3 minutes after time.
- Do not somke and eat for 20 minutes prior testing.
- Do not use any excitant substances near the tester, such as paing, insecticides, alcohols.
- If the device has been performed for five times successively, please stop for two minutes and then use it again.
- Do not drive after drinking!

OPERATION




Different country alcohol units

No.	BAC (Blood Alcohol Concentration)				BrAC (Breath Alcohol Concentration)	
	%	‰	g/l	mg/100ml	mg/l	ppm
1	0.01	0.1	0.1	10	0.05	25
2	0.02	0.2	0.2	20	0.10	50
3	0.03	0.3	0.3	30	0.15	75
4	0.04	0.4	0.4	40	0.20	100
5	0.05	0.5	0.5	50	0.25	125
6	0.06	0.6	0.6	60	0.30	150
7	0.07	0.7	0.7	70	0.35	175
8	0.08	0.8	0.8	80	0.40	200
9	0.09	0.9	0.9	90	0.45	225
10	0.10	1.0	1.0	100	0.50	250
11	0.11	1.1	1.1	110	0.55	275
12	0.12	1.2	1.2	120	0.60	300
13	0.13	1.3	1.3	130	0.65	325
14	0.14	1.4	1.4	140	0.70	350
15	0.15	1.5	1.5	150	0.75	375
16	0.16	1.6	1.6	160	0.80	400
17	0.17	1.7	1.7	170	0.85	425
18	0.18	1.8	1.8	180	0.90	450
19	0.19	1.9	1.9	190	0.95	475
20	0.20	2.0	2.0	200	1.00	500

Common units: %BAC—USA
 g/l—European
 mg/L BRAC—Japan
 mg/100ml— China
 ‰BAC—Russina
 ‰prom—poland

CE RoHS FC FDA MADE IN CHINA

USER'S MANUAL



Digital Display Alcohol Concentration Test Meter

Model: AT7100

CATALOGUE

- Precautions02
- Preparation before using03
- Components diagram04
- Installing battery04
- How to use the alcohol tester05
- Test06
- Auto power off07
- Restart mode07
- Set unit result display format08
- Set alert level.....09
- Rec record checking mode09
- Declaration10
- Common faults and10
- Different country alcohol units11
- Maintenance12
- Performance parameter12

COMPONENTS DIAGRAM

Installing Battery

- Open the back cover
- Install here 1.5V AAA alkaline batteries or two batteries with same specification into the battery case in a right way (a s shown in figure 1) per the mark "+", "-" and "-" on the case, and then close the cover well (as shown in figure 2)
- Install the back cover
- Battery Replacement

If the Battery symbol on the LCD flashes, it indicates the battery electricity is not enough and the batteries need to be replaced. In order to ensure the test accuracy, Open the battery cover and take out the batteries, and reinstall the new batteries with same specification (as shown in figure). Make sure to keep the old batteries out of children's reach, please dispose the useless batteries according to the local regulations.

Set alert level

1. Double click the (K1) button in the TESTING mode to go into CAUTION setting mode, as shown the following picture.

2. Set up CAUTION first, then click (K1) and POWER to set up CAUTION data.

3. After set up the CAUTION data, long press POWER button to switch to DANGER setting, then click K1 and POWER to set up the CAUTION data.

4. After finish setting, double click POWER (K1) button to quit the CAUTION mode, and return back to UNIT SETTING mode.

MEMORY record checking mode

1. Within the TESTING mode time, double click POWER button to get into the RECRcord checking mode, click the left and right button of K1K2 to check the 1-32 times TESTING record.

2. In REC mode, double click POWER button to quit the checking mode and return back to TESTING mode.

3. If there is no any operation within 15 seconds it will show "OFF" to shut off the product.

Maintenance

- Do not use these cleansers such as benzene, thinner or petrol to clean the product to avoid damaging its surface.
- Do not fall it down or strike, otherwise malfunction may be caused.
- Do not try to repair, rebuild or dismantle any parts except for the battery replacement.
- When not use the device for a long time, you are recommended to take out the batteries to prevent the damage due to battery leakage.

Performance Parameter

- Measure range:
 - 0.000-0.190%BAC;
 - 0.000-1.90%BAC
 - 0.000-1.90g/l;
 - 0.000-0.950mg/l
 - 0-190 100mg/100ml
 - 0.000-1.90%Prom
- Alarming Level: 0.05%BAC; 0.5%BAC; 0.5g/l; 0.25mg/l 50mg/100ml 0.5%Prom
- Accuracy: ±0.01%BAC; ±0.1%BAC; ±0.05 mg/l
- Figure size: Length 120mm
- Width 45mm
- Height 20mm
- weight: 38g
- Battery: 3×1.5V(AA A battery)
- blowing Time: 5s
- Display: LCD Display with backlight
- Current consumption: working currents180mA
- Quiescent Currents40μA
- Environmental temperature: -10℃~50℃
- Environmental humidity: ≤95% NO Dews
- Battery expectancy time:≥300

Set unit result display format

Explanation for the 6 kinds of units in the sheet.

value	Meaning
0	%BAC
1	%BAC
2	Mg/L
3	g /l
4	Mg/100ml
5	%Prom

Setting method

1. In the TESTING mode after turn on the product, single click (K1) within the 4 seconds of displaying "WAIT", so as to go into the POINTSETTING mode. Please check the following picture.

2. Then click (K1) to select the unit you need, after set up the unit, Press power to quit the UNIT SETTING mode and return back to TESTING mode.

3. If there is no any operation within 15 seconds it will show "OFF" to shut off the product.

How to use the alcohol tester

Operational Procedure:

- Wait
- Press the power button about 2 seconds, the alcohol tester LCD will turn on with one beep sound.

The "WAIT" symbol is displayed and the 20 to 0 countdown is started that indicate the testers in work up mode.

NOTE

- If first work up is defeated, the tester will be reset automatically and work up again for 1-2 times.
- It always has a large number of gas & impurity adsorbed on the surface of alcohol sensor, when first time taking out the tester from the packaging, or after long time no using for the tester, it need more time for cleaning and work up the alcohol sensor, you should depress the power button and retry several times. If possible.

Set unit result display format

Setting method

1. In the TESTING mode after turn on the product, single click (K1) within the 4 seconds of displaying "WAIT", so as to go into the POINTSETTING mode. Please check the following picture.

2. Then click (K1) to select the unit you need, after set up the unit, Press power to quit the UNIT SETTING mode and return back to TESTING mode.

3. If there is no any operation within 15 seconds it will show "OFF" to shut off the product.

How to use the alcohol tester

Operational Procedure:

- Wait
- Press the power button about 2 seconds, the alcohol tester LCD will turn on with one beep sound.

The "WAIT" symbol is displayed and the 20 to 0 countdown is started that indicate the testers in work up mode.

NOTE

- If first work up is defeated, the tester will be reset automatically and work up again for 1-2 times.
- It always has a large number of gas & impurity adsorbed on the surface of alcohol sensor, when first time taking out the tester from the packaging, or after long time no using for the tester, it need more time for cleaning and work up the alcohol sensor, you should depress the power button and retry several times. If possible.